# **PRODUCT** SUSTAINMENT PACKAGE

# INNOVATIVE TACTICAL TRAINING SOLUTIONS

Maintainance for your industry-leading Tactical Operations Medical Manikin

# KEEPING YOUR EQUIPMENT AT ITS BEST SO YOURE PREPARED FOR THE WORST

Maximize the life of your industry-leading *TOMManikin*<sup>™</sup> by having our expert technicians keep your investment in top-working condition. Let our team keep your equipment running, so you can focus on running your team.

The ITTS Product Sustainment Package includes:

- On-site technician visit
- Complete system inspection, validation
  and function checks
- Software upgrades (main control box, ITTS remote, audio system)
- Inspection of upper airway, lungs, head and neck assemblies
- Upgraded tension pneumothorax system
- Basic on-site repairs

517

• 2.5 hour refresher training

For a complete list of services provided, see reverse.







EMAIL: INFO@TOMMANIKIN.COM

© 2016 Innovative Tactical Training Solutions. All rights reserved.



# INCLUDED SERVICES

# Air Cascade Assembly

Perform visual inspection and function check of scuba/SCBA yoke and supply tubing.

# **Main Control Box**

Visual inspection of box integrity, inspection and verification of each external port connection: air input, tension pneumo, left and right lung, trachea, audio, pulses and bleeding system power supply, power on/off and 'Activate' button function check, air hose port, air regulator gauge, battery connection port.

# Pelvis

Perform visual inspection of steel frame, welds and upper body joint connection assembly. Assess integrity of pelvis exterior foam/silicone.

# Lower Extremities

Perform visual assessment of skin integrity (wounds if applicable), inspect steel hip joint connection point/weld/ pin, perform bleeding function check and tourniquet feature (if applicable).

# Air

Visual inspection and function check of air control manifold, air regulator gauge, brass air pressure regulator (Gen 3 and older), air bottle (pressure gauge, connectors, cascade tubing/connector, supply tubing/connector, and hydro test date), verify/adjust PSI settings of regulator valve.

CALLNOW

**TOLL FREE** 1.866.839.3455

# **Upper Extremities**

UNITED TY

Perform visual inspection of shoulder joint connections, integrity of skin and function check of radial pulse (healthy arm, GEN4 TOMM only), IV tubing and connectors. Perform bleeding function check and tourniquet feature (if applicable).

Kp S()

### **Breather Component**

AND IN THE OWNER OF

Visual inspection and full function check of air control manifold; air input, left and right lung ports, tension pneumo and ports, lung connectors, tubing and lung pillow.

#### Bleeding

Bleeder bag; verify bag integrity, inspect pump power supply cord and connection, bleeder pump function check, priming system tubing and valves, bleeding output tubing to manifold system, inspection and function check of solenoid and regulator valve assembly.

#### Head/Neck

Visual inspection of head skin, eyes (int. eye socket magnets for Gen 4 TOMM), speakers in head assembly, audio cables and connections, teeth, tongue and trachea assembly. Perform function check of carotid pulse (GEN 4 TOMM only), pulse power cable/connector and verification of air from trachea on TOMM exhalation.

# Chest

Perform a visual inspection of interior steel frame, upper extremity joint connections and welds. Inspect/replace tension pneumo bag, sternal I/O assembly, and chest tube assembly, if installed; chest/abdominal wounds, bleeding tubing and connectors.

#### **Battery and Charging System**

Perform visual inspection of all power unit components, connectors, electrical cords, connectors, and LED light. Perform function check on battery and charging system. Replace battery, and/or charger, as necessary. - Replacement battery will be identical amperage. -



# EMAIL: INFO@TOMMANIKIN.COM